**Manufacturer**  
**Vector Surgical**  
 Fabricant  
 Produttore  
 Hersteller  
 Fabricante  
**20975 Swenson Drive, Suite 430**  
**Waukesha, WI 53186 USA**  
**Phone: +1 (262) 798-7970**  
**www.VectorSurgical.com**

**Sterilized using irradiation**  
 Stérilisé par irradiation  
 Sterilizzato tramite irraggiamento  
 Sterilisiert durch Bestrahlung  
 Esterilizado con radiación

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### INSTRUCTIONS FOR USE

#### INTENDED USE

For orienting and labeling excised tissue. Ex vivo use only.

#### INDICATIONS FOR USE

Complex surgical specimens such as resections of breast, pancreas, liver, head and neck, colon, vulva, and soft tissue. Skin resections for basal cell carcinoma, squamous cell carcinoma, and melanoma.

#### PROCEDURE

##### Prepare Specimen

- Open kit within 10 minutes of use.
- Grasp partially excised tissue with clamp to maintain orientation. Complete excision. Blot tissue using clean cloth to absorb excess moisture.

##### Apply Inks to Indicate Tissue Margins

- Apply first color of ink to anterior margin, using large or small applicator. Apply ink in vertical, dabbing motion while securing tissue with clamp. **See #1.**
- Use a different color and a clean applicator for each margin. Apply inks to medial, lateral, inferior, and superior margins.
- After 15 seconds, hold fixative spray bottle a minimum of 6 inches from tissue, and lightly apply 1-2 sprays of fixative to tissue surface where ink is applied. **See #2.**
- After 15 seconds, blot once with clean cloth. (Do not blot more than once.)
- Reposition specimen and apply ink to posterior margin. Wait 15 seconds and then lightly spray fixative on posterior margin.
- After 15 seconds, blot posterior margin once with clean cloth.




##### Apply Clip to Indicate "Area of Interest"

- Identify an area of the specimen for focused attention in pathology analysis.
- Open clip and push firmly into tissue. **See #3.**
- Lock clip by pressing handles together.
- Indicate the reason for marking this area of the specimen on the label provided.

##### Apply Labels and Send Specimen

- Tissue can be placed in preservative immediately after the final blot. Do not leave the inked tissue dry for more than 60 minutes.
- Place labels on container and requisition sheet. Send to pathology.

Note. Do not touch ink to patient. Dispose of after use. Not for internal use.

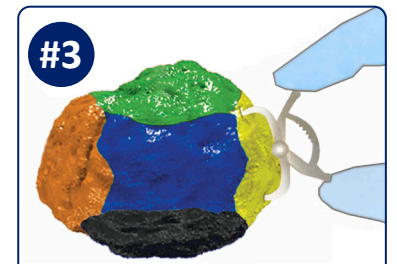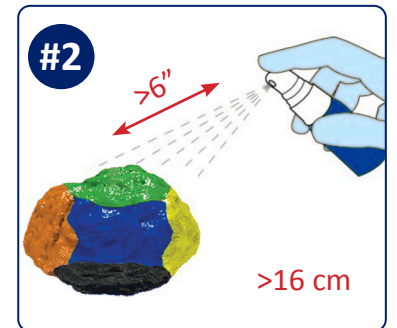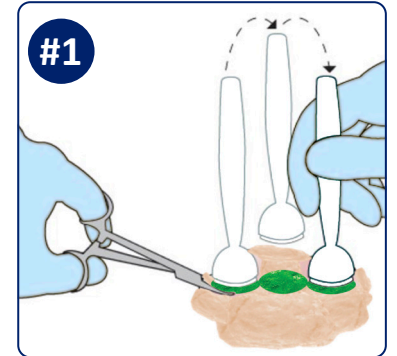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